

# Clean

## Mild Alkaline Detergent

ID 4695

### Description

Clean is specially formulated for use in washer disinfectors. Compatible with a variety of substrates, stainless, aluminum, and plastics. Extremely free rinsing. Strong chelating additives helps protect instruments by preventing scale buildup, allowing hard water minerals to be completely rinsed away. For heavily soiled or stained stainless steel surgical instruments, use Clean undiluted to help in the removal of bioburden and stains. For heavily soiled instruments a pretreatment with Treat Enzymatic Pretreat should be used for the rapid breakdown of bioburden.

### Directions

Clean can be used in mechanical washing systems by injecting into the wash cycle. Typical usage 1/4 to 1 ounce per gallon of water (2 to 8 mL per liter). Clean is not a sterilant. Instruments must be sterilized following treatment with this product.

### Technical Data

Form .....	Liquid	Scent .....	Mild Surfactant
Color .....	Clear	pH (1% solution) .....	8-9



#### WARNING

**HAZARDS:** Causes serious eye irritation.

**PRECAUTIONS:** Wash hands thoroughly after handling. Wear eye protection.

**RESPONSE/FIRST AID:** Have product container or label with you when calling poison control center or physician. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a poison control center or physician for treatment advice. **IF SWALLOWED:** Rinse mouth with water. Contact physician for treatment advice if feeling unwell. **IF ON SKIN (or hair):** Rinse skin with soap and plenty of water. Contact physician if irritation occurs.

**STORAGE & DISPOSAL:** No GHS phrases apply. Dispose of contents/container in accordance with all local, regional, national and international regulations.



Manufactured for:

Spire Integrated Solutions  
A Chalmers Group Company

8719 S 135th St, Ste #300, Omaha, NE 68135 | (402) 344-4200