

DID YOU KNOW?

NO: 151

MARCH 21, 2012

MOVING SETPOINTS FROM AN OLD CONTROL BOARD TO A NEW CONTROL BOARD

It is possible to move all the cycle setpoints from an old control board to a new control board. The procedure is as follows:

Remove the U2 chip from the old control board. The U2 chip is located on the small board (SBC196) on the main control board. The U2 chip is an 8-pin chip

Turn off the power on the new board

Remove the U2 chip and set it aside

Install the U2 chip from the old board on the new board

NOTE

MAKE SURE TO ENSURE THE NOTCH IN THE CHIP MATCHES THE NOTCH ON THE BOARD

Switch the OP CAL switch to OP

Turn on the sterilizer and check the setpoints

Turn the power off and place the OP CAL switch in the CAL position

Turn the power on and calibrate the RTDs and pressure

Turn the power off and move the OP CAL switch to OP

Turn the sterilizer on and run test cycles

Page 1 of 1



PRIMUS

STERILIZER Company, LLC • 117 South 25th Street • Omaha, NE 68131 • Phone (402) 344-4200 • Fax (402) 344-4242