

DID YOU KNOW?

NO: 293

FEBRUARY 4, 2015

HOW TO PRINT OUT ALL PSS8 SETPOINTS (TRINITY)

To print out the PSS8 Trinity Cycle Parameter Settings, do the following steps in sequence:

1. Turn ON the sterilizer, if it is not already on, by using the yellow toggle switch on the control box
2. Wait until the PRIMUS Logo appears on the screen
3. Select the "STER ON" button to go to the DOOR STATUS screen
4. Select the LOGIN button on the DOOR STATUS screen to go to the LOGIN screen
5. Log on to the sterilizer using an ID and password for any of the user IDs 1 - 9. Only user IDs with values lower than 10 (e.g., Administrators and Supervisors) will have access to the necessary menus and functions
6. Select the RETURN button to return to the DOOR STATUS screen
7. Select the MAINT ON button to go to the MAINTENANCE screen
8. Select the VALID CYCLE button to go the VALID CYCLES screen
9. Select the PRINT button at the bottom of the screen to print all the cycle parameters for all valid cycles and the global parameters. Store the printer tape in a cool dry place for later reference
10. Select the RETURN button to return to the MAINTENANCE screen
11. Select the MAINT OFF button to return to the DOOR STATUS screen

For any questions about this DYK, please e-mail Kris Hanson, PRIMUS Manager Controls Engineering, at khanson@primus-sterilizer.com.

Page 1 of 1



PRIMUS

STERILIZER Company, LLC • 6565 South 118th Street • Omaha, NE 68137 • Phone (402) 344-4200 • Fax (402) 344-4242