

DID YOU KNOW?

NO: 60

June 21, 2010


STERILIZER OPTIONS - THE WATER IN DRAIN SYSTEM

Purpose The Water in Drain feature provides more accurate operation of the Drain Temperature Sensor

Parameters N/A

Operation A sight glass in the chamber drain shall be provided for visual monitoring of the drain line for accumulation of condensate and for a water-in-drain sensor. This sensor provides an input to the control system that, when activated, provides for a controlled removal of condensate through the use of a special bleed valve

Exceptions When water is detected in the drain location of the temperature sensor, an additional bleed path is opened to eliminate condensate that may cover the sensor

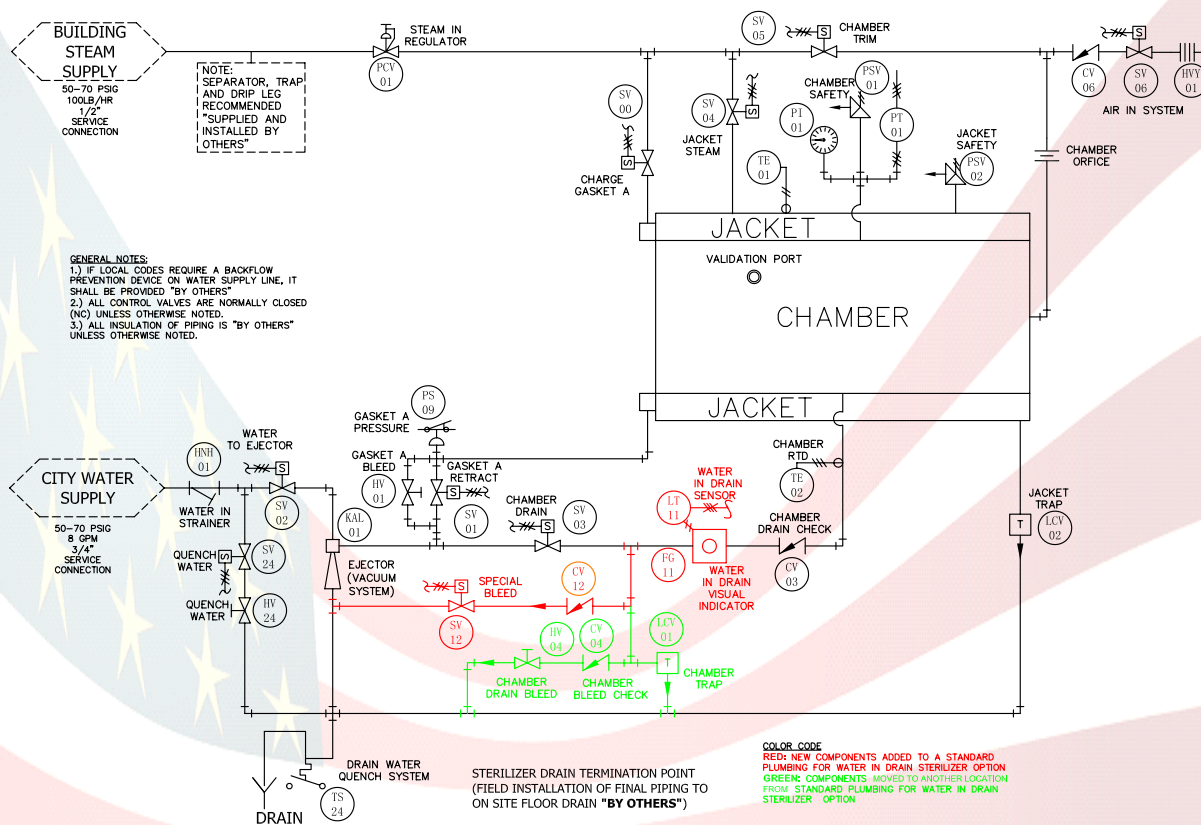

PRIMUS