

Ease



Instrument Lubricant

#4732

Description

A gentle formulation enhanced specially enhanced with non-aggressive ingredients. Frees box locks and lubricates moving parts of surgical instruments. Regular use of Ease protects against corrosion and is compatible with stainless steel, aluminum and a variety of plastics. Contains a preservative to extend shelf life. Ease Instrument Lubricant is not a sterilant. Instruments must be sterilized following treatment with this product.

Directions

Dilute between 0.1 oz/gal (0.8 mL/L) to 0.25 oz/gal (2 mL/L) in a soaking tray, preferably stainless steel. Soak pre-cleaned instruments for 30 seconds. Remove and let drain. Follow up with sterilization. Refer to manufacturer's directions for the type of automatic washer being used. Keep air away from bath by covering tray when not in use. Bath replacement depends upon volume of instruments processed.

For manual applications, this product should be mixed 1:6 with deionized water. Diluted solutions should be discarded after each day.

Technical Data

Form.....	Liquid	pH (1% solution).....	5-6
Color.....	Opaque, White	Viscosity	Thin
Scent.....	Light Sweet Odor	Density.....	8.3 lbs./gal
Foam.....	None	Phosphated.....	No

WARNING

HAZARDS: Causes eye irritation.

PRECAUTIONS: Wash hands thoroughly after handling.

RESPONSE/FIRST AID: Have product container or label with you when calling poison control center or physician. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a poison control center or physician for treatment advice. **IF SWALLOWED:** Rinse mouth with water. Contact physician for treatment advice if feeling unwell. **IF ON SKIN (or hair):** Rinse skin with soap and plenty of water. Contact physician if irritation occurs.

STORAGE & DISPOSAL: No GHS phrases apply. Dispose of contents/container in accordance with all local, regional, national and international regulations.



Manufactured for:

Spire Integrated Solutions
A Chalmers Group Company
8719 S 135th St, Ste #300, Omaha, NE 68135 | (402) 344-4200